SensPERT  
**Canine Heartworm Test Kit**  
*(Dirofilaria immitis)*

**\* Principles**

SensPERT Canine Heartworm Test Kit is designed to detect the antigens of canine heartworm (*Dirofilaria immitis*) in canine whole blood, serum or plasma. Two antibodies in the kit specifically bind with different epitopes of the antigens. After being absorbed into the cellulose pad, the antigens of canine heartworm move and bind with gold-colloid complex of monoclonal anti-canine heartworm of the conjugate pad, forming Ab-Ag complex. This complex, then, forms Ab-Ag-Ab direct sandwich binding with the antibody of another anti-canine heartworm in the nitrocellulose membrane. The test results can appearance on Control and Test lines where the principles of immunochromatography are used.

**\* Characteristics**

- 1) One-step rapid test of canine heartworm antigens
- 2) Rapid test results between 5 ~ 10 minutes
- 3) Expensive equipment not required
- 4) Easy storage and maintenance
- 5) High-purity and high-quality materials of the test kit increase its sensitivity and specificity.

**\* Materials (10 tests/kit)**

- 1) Test ..... 10 units
- 2) Diluent (buffer) ..... 3 ml x 1 unit
- 3) Disposable dropper for sample collection ..... 10 units

**\* Composition**

Specimen well (S : for dropping), Test line (T), and Control line (C) are marked on the device. Inside of it, the strip is composed of sample pad, conjugate pad, nitrocellulose membrane (test paper) and absorbent pad

**\* Effect**

Detection of Canine *Dirofilaria immitis* antigens from canine whole blood, serum or plasma

**\* Uses**

- 1) Specimen  
Canine whole blood, serum or plasma

**2) Test procedure**

- a) When specimen and test kits are stored at cold circumstances (2~8°C), put them at room temperature for 15~30 minutes before use.
- b) Take out a device from a pouch and place it on a horizontal surface.
- c) Using a dropper as a pipette, obtain the specimen and dispense 1 drop (40 µl) of the fluid into the specimen well(S).
- d) When the specimen is completely absorbed into the specimen well(S), drop about 2 drops (80 µl) of buffer.
- e) Read the test results between 5 ~ 10 minutes.

◆ Invalid result after 10 min

**3) Interpretation of the results**

A purple band should appear on the control line regardless of the test result. The presence of another band on the test line determines the result.

Control Line (C): The line should always appear regardless of the presence of the antigens of canine heartworm. If this line does not appear, the test should be considered invalid. This might be because of

impure buffer or the lack of specimen. It should be tested again with another kit.

Test Line (T): The presence of antigens of canine heartworm determines the test line.

- 1) Negative : Control line only appears.

- 2) Positive : Both Test and Control lines appear.

- 3) Retesting

- a) Both Test and Control lines do not appear.

- b) Test line only appears.

**4) Further examinations**

This test is for primary screening only. Consult veterinarians for further necessary examinations to obtain clinical test results.

**\* Precautions**

- 1) Use for canine *in-vitro* diagnostic purposes only.
- 2) Use within 10 minutes after opening the pouch because the test kit is very sensitive to moisture and its effect may diminish.
- 3) Be careful of not touching the result window.
- 4) Every specimen should be used with different droppers.
- 5) For testing, the buffer included should be used.
- 6) Do not use specimen showing hemolysis or being contaminated with microbes, which may cause false positive or false negative result.
- 7) Deal with specimen carefully. They can deliver unknown virus or Infectious bacteria.
- 8) Use disposable gloves when you suspect the infection caused by specimen. And wash your hands later.
- 9) Dispose solid wastes after sterilizing them at 121°C for over 1 hour.
- 10) Do not use the kit when its pouch is torn, sealing is not good in shape or expiration date is passed.

**\* Storage method and expiration date**

The test kits stored at 2~30°C can be used for 24 months after manufacturing. **Do not keep them in a refrigerator.** However, if they are stored under cold circumstances, keep them at room temperature for 15 ~ 30 minutes before use.

**\* Exchange**

The test kits are manufactured under strict quality control system. Nevertheless, if they are deteriorated during delivery, ask our distributors for exchange.

**\* Liability**

The entire risk due to the performance of this product is assumed by the purchaser. The manufacturer shall not be liable for indirect, special or consequential damages of any kind resulting from use of this product.

Date issued : Sep. 08, 2016  
 AI01-01E-03

**VETALL**  
 Manufactured by  
**VetAll Laboratories**  
 #B-305, Samsong Techno Valley, 140, Tongil-ro,  
 Deogyang-gu, Goyang-si, Gyeonggi-do, 10594, Korea  
 Tel : 82-2-22193459, Fax : 82-2-22193457

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